



## PPE - Face Shields and Goggles


### Cleaning and Disinfection Instructions

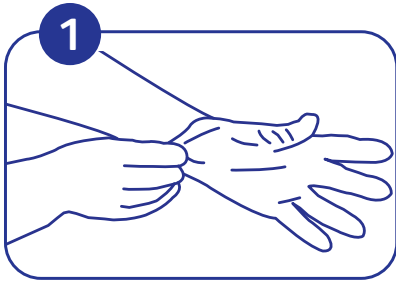


Don fresh gloves.

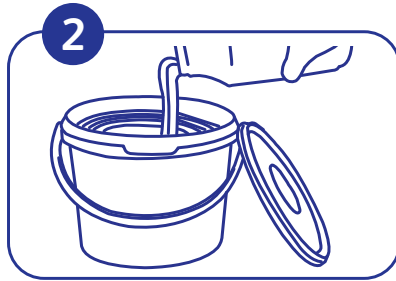


Pour disinfectant use solution in EasyWipe™ bucket. Allow to soak in before replacing lid.

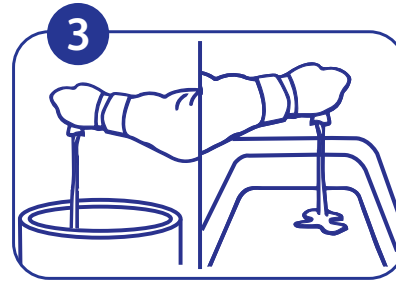


Express excess liquid from wipe.

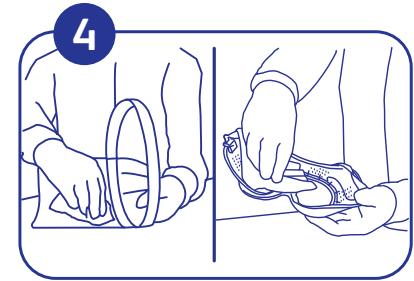


Wipe inside of face shield / goggles.

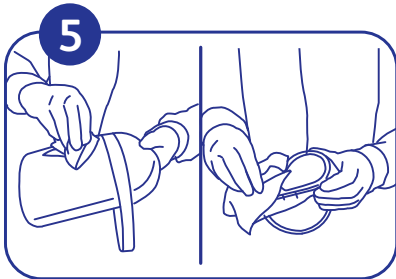

Wipe outside of face shield / goggles.

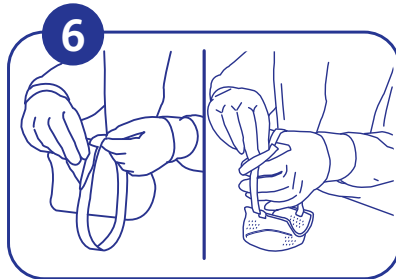

Wipe strap of face shield / goggles. Allow all to remain wet for the contact time on the disinfectant label.

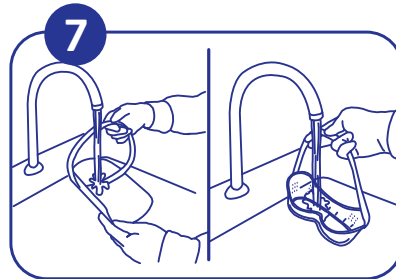


Rinse and allow to air dry.

**Questions: 1-800-558-2332**


Always wear personal protective equipment.

# PPE - Face Shields and Goggles


## Cleaning and Disinfection Instructions

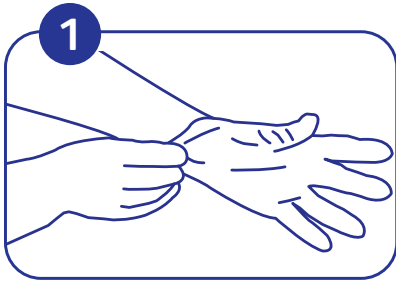


Don fresh gloves.

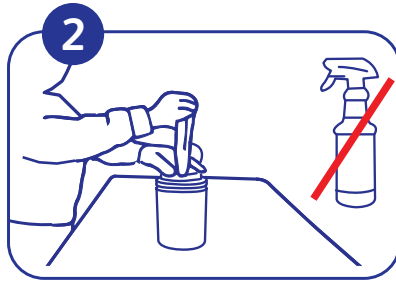


Dispense disinfectant wipe.  
Close lid.

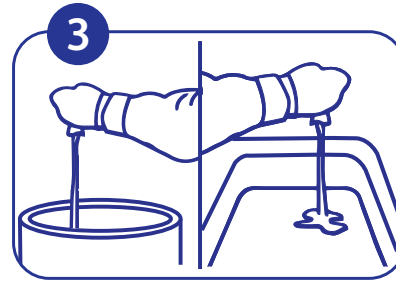


Express excess liquid from wipe.

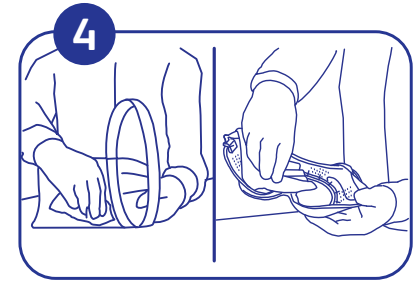


Wipe inside of face shield /  
goggles.

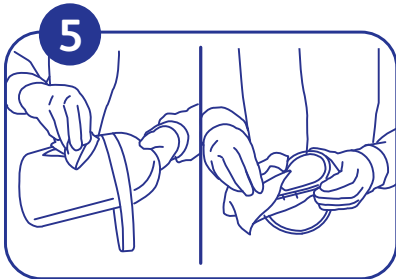

Wipe outside of face shield /  
goggles.

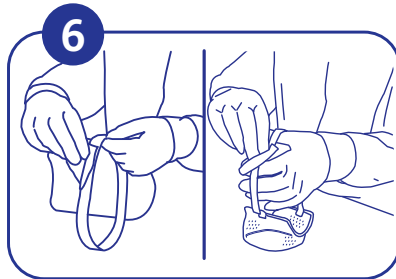

Wipe strap of face shield /  
goggles. Allow all to remain  
wet for the contact time on the  
disinfectant label.

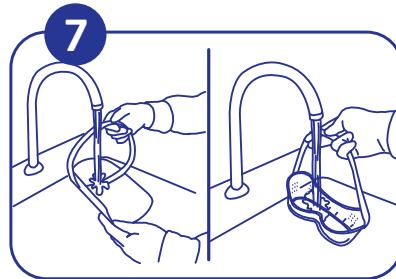

Rinse and allow to air dry.

**Questions: 1-800-558-2332**

Always wear personal protective equipment.