

Feel free to touch

Safety is just 1 wipe away



Why should you use wipes?



Full virucidal efficacy in **just 30 sec**
(Coronavirus and Norovirus)



Effective against Bacteria, Viruses
and Yeasts



Easy to use by **anyone, anytime, anywhere**



AHP is **faster, safer and more sustainable** than other actives



No time needed to prepare and use



Safe for users, no need for PPE



360° protection for your staff,
visitors and facilities



Improves **labour efficiency by 23%**
vs. traditional bucket and cloth*



Proven **increase of disinfection compliance** by 35%*



Avoid & prevent
cross-contamination of surfaces



Non flammable



Compatible with most hard surfaces



Avoid laundering cloths



Eliminates risks of spills & trips

* Results from a study published on the American Journal of Infection Control.

Use biocides safely. Always read the label and product information before use.

THINK

Every frequently touched
surface needs to be wiped

www.diversey.com

These materials are provided for general information purposes only and do not replace each user's responsibility to assess the operational, legal and other requirements applicable to each facility.

All logos and trademarks are owned by or licensed to Diversey, Inc. © 2021 Diversey, Inc. All Rights Reserved. 88006 MASTER 01/21

Diversey
OxivirTM
powered by **AHP**[®]

Feel free to touch

Safety is just 1 wipe away

LONG TERM CARE

When to wipe



Before/after aseptic procedures (e.g. wound dressing/CL/PICC Insertion, etc.)



Before placing a food tray



After assistance with productive coughing or vomiting




When surfaces are visibly soiled




After patient bathing within bedspace or after any procedure involving feces/bodily fluids/respiratory secretions


How to wipe

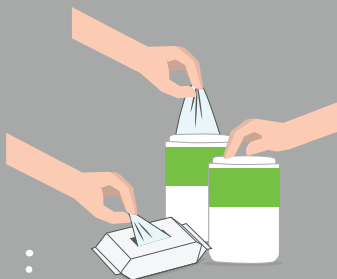


Take a wipe from the dispenser and close the pack firmly to prevent remaining wipes from drying out

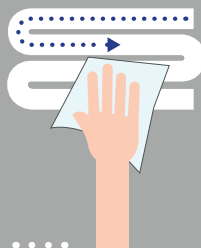

Holding the wipe flat in your hand (not scrunched), wipe the surface in an 'S' shape, moving from clean to dirty. Do not go over the same surface twice with the same wipe

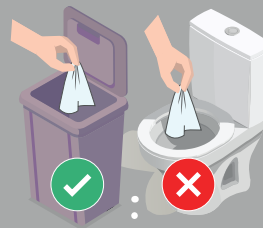

Dispose of used wipes in the appropriate waste bin following local policy guidelines. Save your wipes, don't flush wipes.

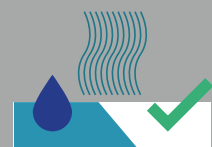

Allow surface to dry before use

THINK

Every frequently touched surface needs to be wiped

www.diversey.com

These materials are provided for general information purposes only and do not replace each user's responsibility to assess the operational, legal and other requirements applicable to each facility. All logos and trademarks are owned by or licensed to Diversey, Inc. © 2021 Diversey, Inc. All Rights Reserved. 88006 MASTER 01/21



Oxivir™

powered by **AHP®**